

Supernal[®]

HI-LOW

ASSEMBLY MANUAL

V2.1




**Transfer[™]
Master**

Celebrating 30 years in business




ASSEMBLY MANUAL

The **Supernal Hi-Low** bed arrives in 2 boxes, one box for the Hi-Low Unit and one box for the bed deck.

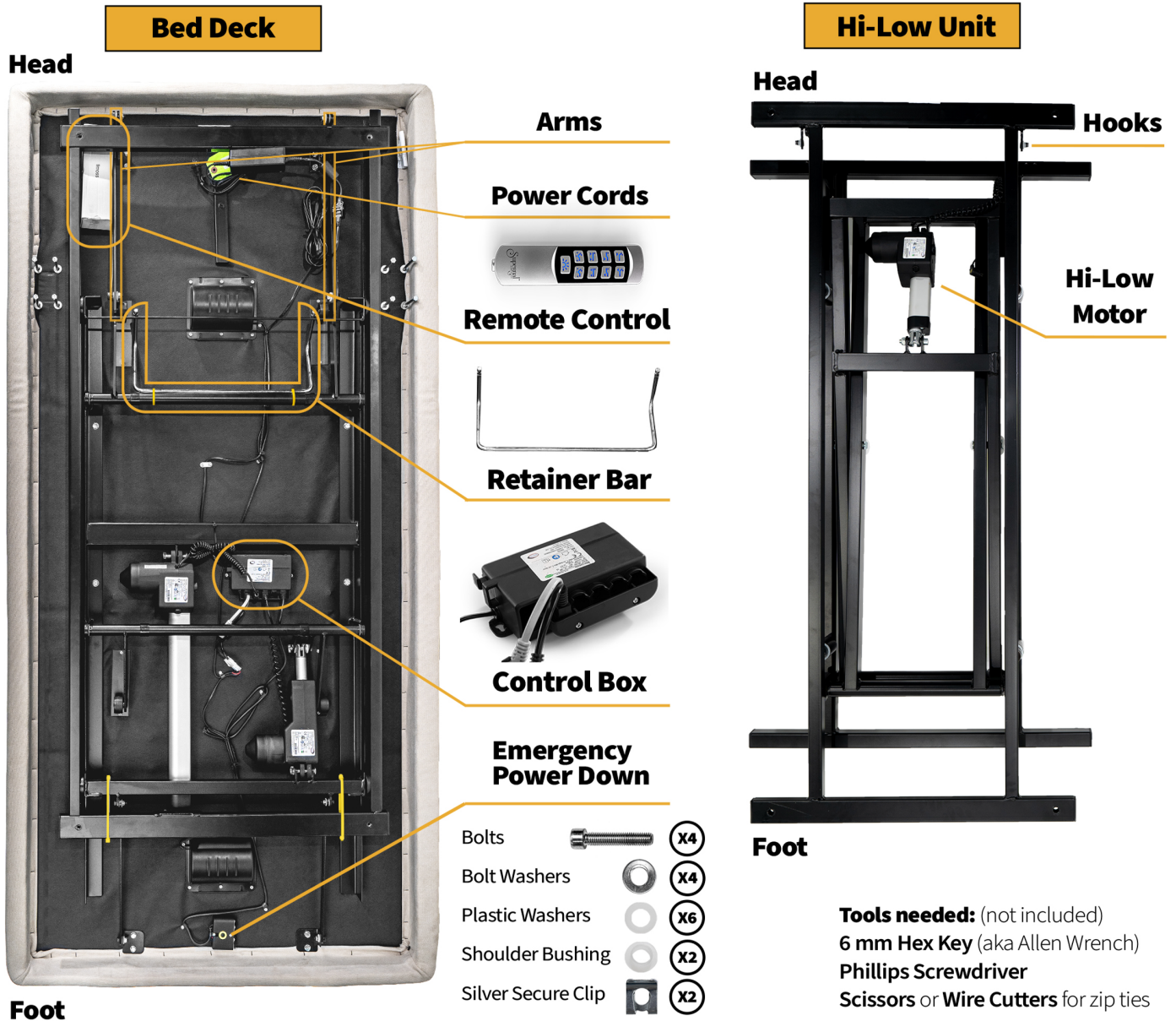

PREPARATION



- Lay the shipping boxes on the floor, side by side.
- Remove the box lids and take out all the accessories and parts.
- Carefully cut all yellow zip ties and free loose components.
- Position the Hi-Low unit next to the bed deck. Pay careful attention that the head sections and the foot sections line up correctly, as shown above.
- Locate the arms at the head of the bed deck and pull them upwards.

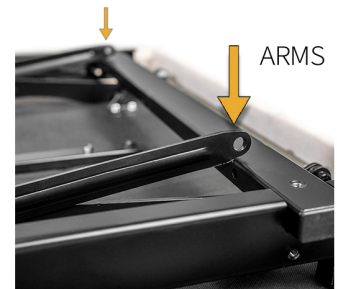

1 ASSEMBLY

Carefully lift and flip the Hi-Low unit over the bed deck, ensuring proper alignment with the pre-drilled holes. Connect the arms that are used for the backrest lifting mechanism to the Hi-Low unit.

Locate the two hooks on the Hi-Low unit that have studs welded to them. Locate the silver secure clip, bushing, and washer from the plastic bag on the bed deck. Assemble the lifted arms with the hooks, as shown below.

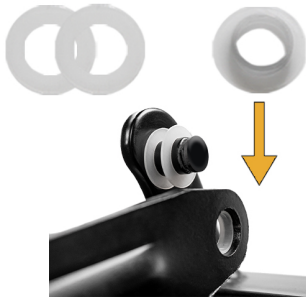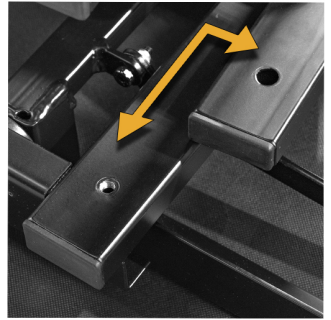

2 WASHERS +
SHOULDER BUSHING



PLACE THE ARM



WASHER



SECURE CLIP



REPEAT THIS ACTION
ON BOTH SIDES
OF THE BED

2 ATTACHING THE HI-LOW TO THE BED DECK

Securely fasten the bed deck and Hi-Low unit together. Place the washer first and then tighten the bolt using a Hex Key. Repeat on all four edges.




3 CONNECTING THE HI-LOW TO THE CONTROL BOX

The control box has a cap to ensure that the wires remain secured. The cap is screwed onto the control box. To access the input and connect the Hi-Low motor to the control box, simply unscrew the cap using a Phillips Screwdriver.

Plug the connector of the Hi-Low motor cable into the free socket on the control box. **Twin size** - 1 cable, **Full** or **Queen size** - 2 cables.

Make sure it has the same orientation as the other plugs already connected. Screw the cap back onto the control box after it's plugged in. At this point two people can flip the bed over.

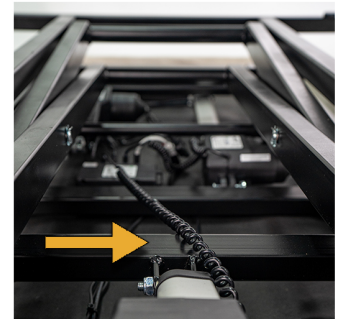

While upside down, make sure the cable from the Hi-Low motor is routed over the bar



BEFORE:
CONTROL BOX WITH
THE CAP



CONTROL BOX CAP



HI-LOW CONNECTOR
CABLE



CONTROL BOX
CONNECTED
WITHOUT THE CAP



AFTER:
CONTROL BOX CONNECTED
WITH THE CAP

4

RETAINER BAR ASSEMBLY

Locate the mattress retainer bar (zip tied to the bed deck in delivery). Slide the retainer bar into the insert and secure with the pin.

Twins have one retainer bar, **Fulls** and **Queens** have two.



The assembly of the **Supernal Bed System** is now complete. Connect the power cord to a nearby outlet and enjoy your new bed!

BED ACCESSORIES ASSEMBLY (ACCESSORIES SOLD SEPARATELY)

GUARD RAILS

ASSEMBLY INSTRUCTIONS

1. Bed must be assembled and upright. Locate the 4 pre-drilled holes on the bed deck.


2. Align the guard rail bracket over the pre-drilled holes with the plastic rail silencer facing upwards. Place the washer over the bolt and thread through bracket. Use a socket wrench to fasten the bolts. Do **not** overtighten them.

3. Pull the bracket pin, insert the guard rail until the pin clicks into place.

Repeat these steps for other side if applicable.

Tools needed:

7/16 socket wrench (not included)

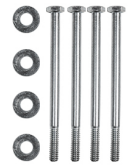

HEADBOARD

ASSEMBLY INSTRUCTIONS

1. Locate the pre-drilled holes on the bottom edges of the bed deck.

2. Align the headboard bars with the pre-drilled holes, place washer on bolt and thread through pre-existing hole. Repeat on the other side.

3. Slide the headboard into the headboard bars.

Tools needed:

6 mm Hex Key (included)



HEADBOARD BARS



HEADBOARD

